

Physician TraceMasterVue ECG - Quick Guide for Over-Reading

Logon to TraceMasterVue

- ☐ Locate TM ECGVUE icon on desktop or locate TraceMasterVue on Citrix
- ☐ Log-on with your MHS/NT username and password

Locate your ECGs to review

- ☐ Select Search Type: **[ECGs assigned to specific inbox]**
- ☐ **Highlight** your inbox in the 'Assigned to' box.
- ☐ Click **[Search]** – A list of ECGs by patient opens
- ☐ Click **[Select All]**. A check appears in box to left of all ECGs
 - o Deselect (Uncheck) any you do not wish to review
- ☐ Click on **[Review]**.
 - o The oldest ECG Statement needing confirmed opens

Interpretations

- ☐ Click in the 'Interpretation' box to add or remove text as desired.
- ☐ Click buttons on right to:
 - o Insert Code, Insert Text, Edit – add modifiers, Delete, Undo
- ☐ Use 'Calipers' as necessary
- ☐ Severity: Click the down arrow to change
- ☐ Click **[Waveform]** to view current and previous ECG side by side
- ☐ Click on **[Confirm]** to confirm ECG
 - o This electronically signs the ECG
- ☐ The next ECG that needs review will appear

When finished with your session, Click **[Logout]** on the Action Bar